



# Entering Room with Personal Stethoscope

- 1 Perform hand hygiene  
→ **Don** gloves

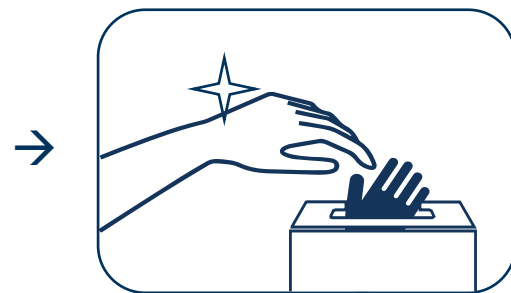


- 2 Use disinfectant wipe to clean stethoscope  
→ Place in **CLEAN** bin

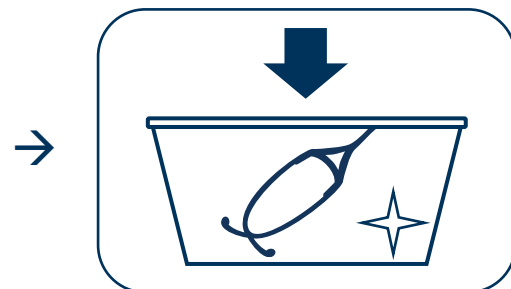



- 3 **Doff** gloves →  
Perform hand hygiene

- ! **Do not put  
stethoscope on your  
neck**

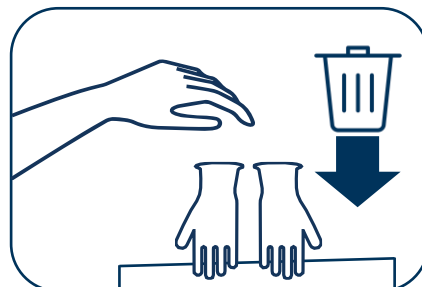


- 4 **Don** PPE → Enter  
room with clean  
stethoscope

- ! **Door knobs are  
dirty. Do not touch  
with clean PPE**

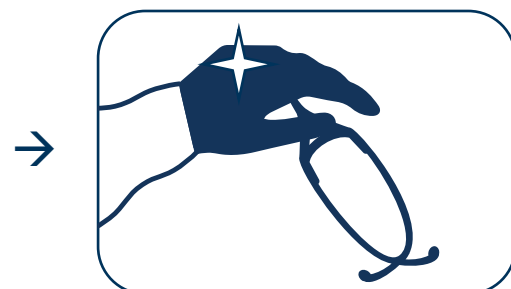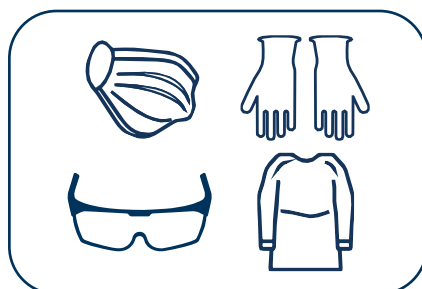


# Cleaning Stethoscope and Reusable Eye Protection After Use

- 1** **Doff** gloves → Perform hand hygiene  
**Doff** gown → Perform hand hygiene

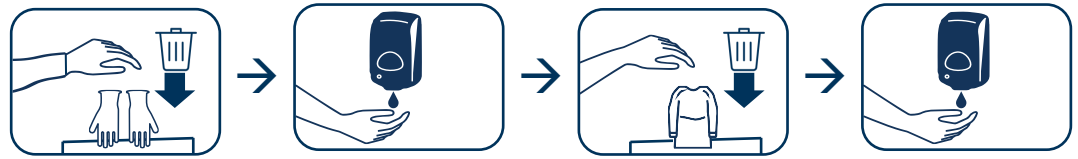


- 2** **Don** clean gloves → Use disinfectant wipe to pick-up stethoscope

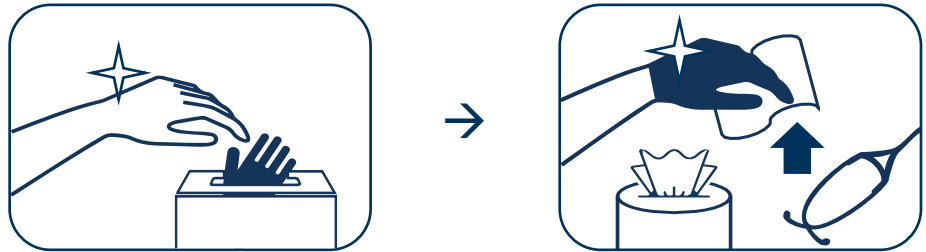

- !** **Door knobs are dirty. Do not touch with clean PPE**

## Exit Room


- 3** Use disinfectant wipe to clean stethoscope → Place in CLEAN bin

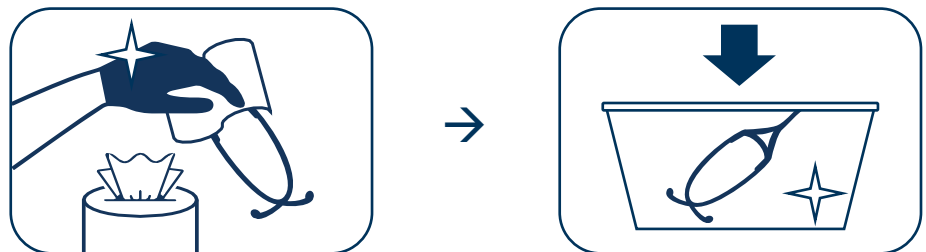


- 4** **Doff** gloves → Perform hand hygiene  
**Don** clean gloves



- 5** Remove and clean eye protection with disinfectant wipe → Place in CLEAN bin


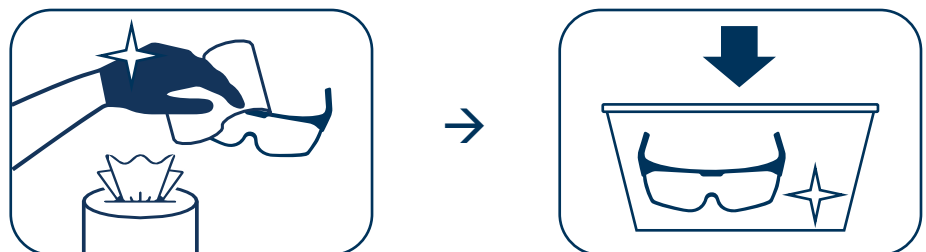


- 6** **Doff** gloves → Perform hand hygiene  
**Doff** mask → Perform hand hygiene

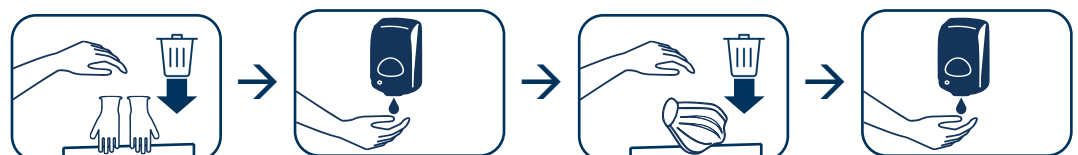


# Cleaning In-Room Stethoscope

- 1** In PPE, grab disinfectant wipe → Use disinfectant wipe to clean stethoscope

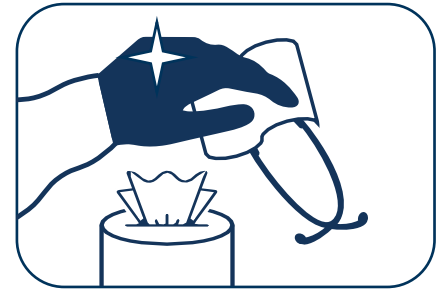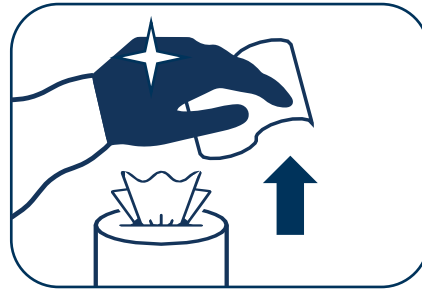



- 2** Place stethoscope on CLEAN surface (e.g. paper towel) and let dry

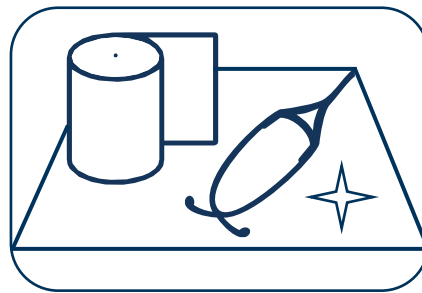



- !** If you are unable to grab a disinfectant wipe without touching the container, Doff gloves in room and Don clean gloves before cleaning stethoscope

## After Stethoscope Use

- 3** Grab new disinfectant wipe without touching container → Clean earpieces

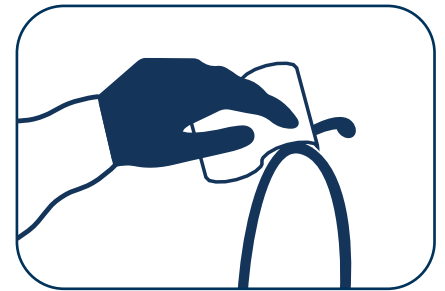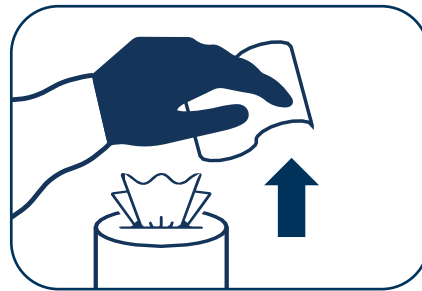

- 4** Place stethoscope on clean hook → Clean downwards

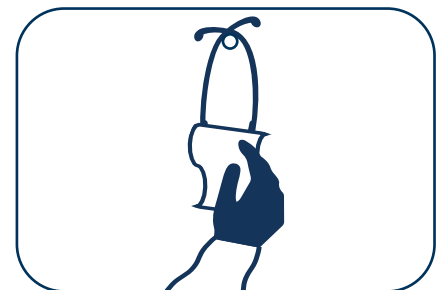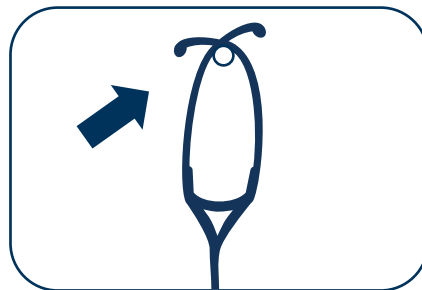

- !** Do not touch cleaned areas with gloves